

## COVID-19 SELF TEST KIT

### Introduction

Online  
Training



The Ministry of Health has certified the Covid-19 self-test kit which was given conditional approval on July 21 to be imported and distributed nationwide.

Minister of Health, Dato 'Sri Dr. Adham Baba said, the kit can now be sold at all registered community pharmacies or health facility premises in the country.

"Those who do the self-test, however, must immediately enter the results received through the MySejahtera application, whether negative, positive or invalid status," he said in a virtual press conference of the Covid-19 Immunization Special Task Force (CITF) with the Coordinating Minister of Covid Immunization Program- 19 National, Khairy Jamaluddin Abu Bakar.

### Complete explanation of the COVID-19 Self-Test Kit Management Guidelines;

The COVID-19 Self-Test Kit was used as a self-administered screening test. The COVID-19 Self-Test Kit does not replace the real-time diagnostic function of reverse-transcription polymerase chain reaction (RT-PCR) for COVID-19 but has the advantage of detecting individual COVID-19 status quickly and easily.

The COVID-19 Self-Test Kit that can be used is one that has received a recommendation from the Medical Device Authority (MDA). The list of test kit types will be updated on the MDA website from time to time.

### The COVID-19 Self-Test Kit can be used to:

- Self-screening by any individual wishing to know the status of COVID-19;
  - Individuals who have had contact with COVID-19 cases; OR
  - Other appropriate screenings (such as International Entrance, cross-state, pre-employment screening, pre-admission to higher institutions etc.)
- The user must ensure that the kit purchased has obtained permission from the MDA.



**Self-Test Covid-19 Test Kit For Conditional Approval (Approved by MDA, MOH)**

Refer to list of tests approved by MDA below. (Updated on 29<sup>th</sup> July 2021)

Below is the list of all tests that have been selected to date and the status is Conditional Approval (please note: list is updated on a routine basis).

Notes:  
\*Sample type is based on testing facility evaluation report.

NO	COMPANY NAME	PRODUCT NAME	MANUFACTURER	IDENTIFIER	DETECTION	SAMPLE TYPE
1	Reszon Diagnostic International Sdn Bhd	Salixium-COVID-19 Rapid Antigen Rapid Test (Saliva/ Nasal Swab Samples)	Reszon Diagnostics International Sdn Bhd. selangor, Malaysia	SLXHB1-0621001	RTK-Antigen (Self-test)	Saliva and nasal swab samples (Mix Samples)
2	Citymedic Sdn Bhd	Gmate® COVID-19 Ag Saliva for Home Use	Philosys Co., Ltd, Korea	AG-020	RTK-Antigen (Self-test)	Saliva
3	Medinics (M) Sdn Bhd	BERIGHT-COVID-19 Antigen Rapid Test Device (Oral Fluid)	Hangzhou AllTest Biotech Co., Ltd.	ICOV-802H	RTK-Antigen (Self-test)	Saliva
4	Global Science Sdn Bhd	ALLTest COVID-19 Antigen Rapid Test (Oral Fluid)	Hangzhou AllTest Biotech Co., Ltd Zhejiang, P.R. China.	ICOV-802H	RTK-Antigen (Self-test)	Saliva
5	Neopharma Biotech Asia Sdn Bhd	JusChek COVID-19 Antigen Rapid Test (Oral Fluid)	Hangzhou AllTest Biotech Co., Ltd Zhejiang, P.R. China.	ICOV-802H	RTK-Antigen (Self-test)	Saliva
6	Jingga Anggun Sdn Bhd	Longsee 2019-nCoV Ag & Influenza A/B Rapid Co-Detection Kit (Immunochromatography_ - Home based Use	Guangdong Longsee Biomedical Co. Ltd., Guangzhou, China	LS-C-T-005	RTK-Antigen (Self-test)	Saliva

Updated 29 July 2021

<https://portal.mda.gov.my/announcement/631-self-test-covid-19-test-kit-for-conditional-approval-approved.html>


One of approved Self test kit is ALLTest COVID-19 Antigen Rapid Test Device (Oral Fluid). All test kit shall be packed completely with user information, manual, and how to read results. Details procedure are also available in video for better understanding.

**ALLTEST**  
DOCTOR ON CALL

**COVID-19 ANTIGEN RAPID TEST KIT (ORAL FLUID)**

**HOW TO USE**

1. Deeply cough 3-5 times. Fit the funnel onto the tube. Spit oral fluid. Remove the funnel. The oral fluid (non-bubble) should just reach the scale line.
2. Place the tube into the tube holder. Add entire disposable Buffer. Squeeze 10-15 times.
3. Add 2 drops of solution.
4. Wait 15 Min. Results: Positive (C and T lines), Negative (C line only), Invalid (T line only).

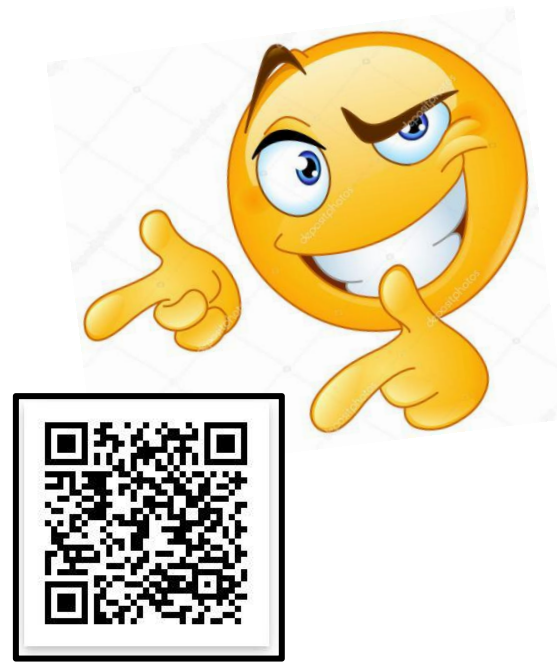

**Reminder !**

Please refer to “Flow Chart of Management of Covid-19 Self Test Kit”. You need to do Self Reporting via MySejahtera.

Make sure to insert your test kit waste properly into Biohazard plastic bag which provided together with set of kit.

Flow Chart of Management of Covid-19 Self Test Kit



Document Reff. : Kementerian Kesihatan Malaysia (KKM)